

RAPID STD TEST (FEMALE)

DESCRIPTION

Knowing your status has never been easier. Visby's Rapid STD Test (Female) product tests for three of the most common - and fully treatable - sexually transmitted infections: chlamydia, gonorrhea, and trichomoniasis. All it takes is a self-collected vaginal swab. You can choose to take the Visby Medical Women's Sexual Health device home and test in the privacy of your own space, or you can choose to perform the test in our store with a trained staff member available to offer assistance and answer questions if needed. When you purchase the Visby Medical Women's Sexual Health device, you get clinically accurate results in only 30 minutes. Your test results will be sent to the Visby Medical App immediately for your review. Just download the Visby app (available on iPhone and Android), create your account, and you're ready to go.

What happens if you test positive? You're not on your own. Same-day, third-party telehealth with a licensed healthcare provider is included, so you can get answers and explore treatment options fast with the click of a button in the Visby Medical App. *(Please note that medication costs are not included.)*

A Few Important Things to Know:

- Any Lab Test Now is partnered with Visby Medical to offer this rapid, accurate test to support female sexual health.
- This is the first at-home PCR test for the three most common, curable STIs, now available at Any Lab Test Now.
- Your results are yours. They will not be shared with or accessible through us without your explicit consent.
- This test is a helpful tool, however, it is not a substitute for visits to a healthcare provider. The result obtained with this test should not be used to start, stop, or change any course of treatment unless advised by your healthcare provider.
- This test is for **female customers only**.

Test Details:

This test includes:

- Chlamydia (*Chlamydia trachomatis*)
- Gonorrhea (*Neisseria gonorrhoeae*)
- Trichomoniasis (*Trichomonas vaginalis*)

WHO SHOULD GET THIS TEST?

This test is for females experiencing symptoms that could be caused by an STI, or who are concerned that they may have been exposed to an STI.

WHAT KIND OF SAMPLE IS REQUIRED?

A self-collected vaginal swab is required for this test.

ARE THERE INSTRUCTIONS FOR COLLECTING THE SAMPLE?

Yes. There is a QR code on the Visby device box to download the Visby App for step-by-step video instructions. There are also printed instructions in the box.

WHAT DO I NEED TO DO TO PREPARE?

Refrain from the use of any of the following for 48 hours before taking this test: douches, washes, lubricants, vaginal wipes, vaginal moisturizers, antiperspirants, deodorants, or feminine hygiene spray. If you are taking the antibiotic doxycycline (Vibramycin), wait until it has been at least 48 hours since your last dose before taking this test.

DO I NEED TO COLLECT MY SAMPLE AT A SPECIFIC TIME OF DAY?

No. You can collect and test at any time of day or night.

CAN I COLLECT MY SAMPLE IF I AM ON MY PERIOD?

You can take this test at any point in your menstrual cycle. However, it is best to avoid testing during the heavy flow day(s) of your period.

HOW LONG WILL IT TAKE TO GET MY RESULTS?

Your test results will be available in your Visby Medical App profile in approximately 30 minutes.

WHAT DOES PCR MEAN? DOES IT IMPROVE TEST ACCURACY

PCR testing is the same gold-standard technology used in professional labs.

IS A DOCTOR'S ORDER REQUIRED?

No. This test is approved for home use without a doctor's order.

IF I HAVE QUESTIONS ABOUT MY RESULTS, WHO SHOULD I CONTACT?

If you've already taken a test and need to speak with a provider, log in to the Visby App and follow the prompts to connect with a licensed third-party telehealth provider.

DO I NEED TO MAKE AN APPOINTMENT?

No. You can walk in and purchase at any time. You can also order and pay through our website to ensure that pickup is quick and easy.

CAN MEN TAKE THIS TEST ALSO?

This test is currently available to female customers only.

AM I REQUIRED TO FAST?

No. Fasting is not required prior to taking this test.

OTHER RELEVANT TESTS

If you are interested in this panel, you may also want to consider:

- Basic STD Panel
- Comprehensive STD Panel
- Comprehensive STD Panel Plus